

Anti-Glypican-3

Catalog No.	Description
AM539-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AM539-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
MU539-UC	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
MU539-5UC	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AX539-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 160 tests
AX539-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 50 tests
AX539-4M	Ready-to-Use Antibody in Barcode labeled vial for use on the NanoVIP® Staining System, 50 tests
AX539-10M	Ready-to-Use Antibody in Barcode labeled vial for use on the NanoVip™ 300 Staining System, 100 tests
VM539-5M	6 ml of Ready-to-Use Antibody for use with BioGenex / equivalent Detection systems on third party automation.

Clone	Species	Ig Class
GPC3-88	Mouse	IgG1/K

Intended Use

For Research use only. This antibody is designed for the specific localization of human Glypican-3 in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

Summary and Explanation

Glypican-3 (GPC3) is a glycosylphosphatidyl inositol-anchored membrane protein, which may also be found in a secreted form. GPC3 belongs to a member of the glypican family of heparan sulfate proteoglycans. This protein may be involved in the suppression/modulation of growth in the predominantly mesodermal tissues and organs. Glypican-3 is thought to regulate tissue and organ growth through interactions with growth factors such as Insulin-like growth factor II or fibroblast growth factor 2. GPC3 was identified to be useful tumor marker for the diagnosis of hepatocellular carcinoma (HCC), hepatoblastoma, melanoma,

and testicular germ cell tumors. A high level of GPC3 expression has also been found in some types of embryonic tumors, such as Wilms tumor and hepatoblastoma.

Storage and Handling

Store at 2-8°C. Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

Reagents Provided

Mouse Monoclonal Antibody to human Glypican-3 is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	Hepatocellular Carcinoma as available with Biogenex FB-539M* & FG-539M*
Recommended Dilution for Concentrated Antibody	1:20-50 in HK941
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK)
Recommended Pretreatment (Xmatrx & NanoVIP)	EZ-AR2 Elegance (HX032-YCD & HX046-08XN)
Antibody Incubation (Manual/i6000)	30-60 min at RT
Antibody Incubation (Xmatrx & NanoVIP)	30-60 min at 25°C

Category	Antibodies	Revision No.	M
Document No.	932-539M-EN	Release Date	05-May-2025

Detection System for Manual, Xmatrx, NanoVIP & i6000 systems***	Use BioGenex Two-Step OR One-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information
---	---

*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx & NanoVIP requires micro chamber slides..

**Pretreatment times will vary based on individual microwave power.

***For automation systems (Xmatrx-Elite, NanoVIP & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Detection System	Two-Step HRP Kit	One-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKEN (1000 Test)	QD630-XAKEN (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKEN (1000 Test)		
	QD420-YIKEN (500 Test)	QD620-XAKEN (500 Test)	QP900-9LE (500 Test)
	QD400-60KEN (60 Test)		
Xmatrx - Automation	QD550-YCDEN (200 Test)	QD610-YADEN (200 Test)	N/A
NanoVIP- Automation	QD551-YCDEN (100 Test)	QD611-YADEN (100 Test)	N/A
i6000 - Automation	QD410-YAXEN (200 Test)	QD610-YAXEN (200 Test)	N/A
For more information, visit www.biogenex.com .			

Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or

contact BioGenex Technical Support Department at 1-800-421-4149 or support@biogenex.com or your local distributor to report unusual staining.

Expected Results

This antibody stains membrane/cytoplasm in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

Bibliography

- Marrero JA. J Hepatol 2009; 50: 659-661.
- Anatelli F, et al. Am J Clin Pathol 2008; 130:219-223.
- Coston WM, et al. Am J Surg Pathol 2008; 32:433-44.
- Kakar S, et al. Arch Pathol Lab Med 2007; 131:1648-1654.
- Zhu ZW, et al. Gut 2001; 48: 558-564.
- Center for Disease Control. Decontamination of Laboratory Sink Drains to Remove Azide Salts. Center for Disease Control Manual Guide--Safety Management, No. CDC-22, Atlanta, Georgia. April 30, 1976.
- Kiernan JA. Histological and Histochemical Methods: Theory and Practice. New York: Pergamon Press 1981.
- Nadji M, Morales AR. Immunoperoxidase, part 1: the techniques and its pitfall. Lab Med 1983; 14:767-770.

	Temperature Limitation	RUO	Research Use only
	Use By Date	LOT	Batch Code
	Non-Sterile		Consult Instructions for Use
	Manufacturer,	BioGenex	

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	M
Document No.	932-539M-EN	Release Date	05-May-2025