

Anti-CD223

Catalog No.	Description
AMD50-5M	6 ml of Ready-to-Use Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AMD50-10M	10 ml of Ready-to-Use Antibody in a barcode labeled vial for use with BioGenex Super Sensitive™ Detection Systems and i6000™ Automated Staining Systems
MUD50-UC	1 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
MUD50-5UC	0.5 ml of Concentrated Antibody for use with BioGenex Super Sensitive™ Detection Systems OR equivalent detection system
AXD50-YCD	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 160 tests
AXD50-50D	Ready-to-Use Antibody in Barcode labeled vial for use on the Xmatrx® Elite Staining System, 50 tests
AXD50-4M	Ready-to-Use Antibody in Barcode labeled vial for use on the NanoVIP® Staining System, 50 tests

Clone	Species	Ig Class
D-8	Mouse	IgG2b, kappa

Intended Use

For In Vitro Diagnostic Use. This antibody is designed for the specific localization of human CD223 in formalin-fixed, paraffin-embedded (FFPE) tissue sections. Evaluation must be performed by a qualified pathologist.

Summary and Explanation

CD223, also referred to as LAG-3, is a 70-kDa surface glycoprotein belonging to the Ig superfamily and sharing homology with CD4. It exhibits a greater affinity for binding to MHC class II than CD4 and primarily functions in the inhibition of T cell activation and homeostatic proliferation. CD223 is expressed on activated T cells, including regulatory T cells and NK cells, with higher levels seen in CD8+ T cells than CD4+ T cells. Co-expression of CD223 and CD49b has been proposed as a marker for identifying Type 1 regulatory T cells (Tr1 cells). CD223's role as a high-affinity MHC class II ligand and its association with Tr1 cells highlights its significance in cancer diagnosis and immunotherapeutic approaches.

Storage and Handling

Store at 2-8°C. Fresh dilutions, if required, should be prepared prior to use and are stable and steady for up to one day at room temperature (20-26°C). Diluted antibody preparations can be refrigerated or frozen for extended shelf life.

Principles of the Procedure

Antigen detection by immunohistochemistry (IHC) is a two-step process wherein the primary antibody binds to the antigen of interest and that binding is detected by a chromogen. The [primary antibody](#) may be used in IHC using manual techniques or BioGenex Automated Staining System. Positive and negative controls should always be run simultaneously with all patient specimens.

Reagents Provided

Mouse Monoclonal Antibody human CD223 is affinity purified and diluted in PBS, pH 7.2, containing 1% BSA and 0.09% sodium azide.

Dilution of Primary Antibody

BioGenex Ready-to-Use antibodies have been optimized for use with the recommended BioGenex Detection System and should not require further dilution.

BioGenex concentrated antibodies must be diluted in accordance with the recommended protocol when used with the recommended BioGenex Detection System.

Recommended Protocol

Refer to the following table for conditions specifically recommended for this antibody. Refer to the BioGenex website for guidance on specific staining protocols or other requirements.

Parameter	BioGenex Recommendations
Control Tissue	Ovary and Tonsil tissues as available with Biogenex FB-D50M* & FG-D50M*
Recommended Dilution for Concentrated Antibody	1:20-50 in HK941
Recommended Pretreatment (Manual/i6000)**	EZ-AR2 (HK522-XAK)
Recommended Pretreatment (Xmatrx & NanoVIP)	EZ-AR2 Elegance (HX032-YCD & HX046-08XN)
Antibody Incubation (Manual/i6000)	30-60 Min at RT
Antibody Incubation (Xmatrx & NanoVIP)	30-60 Min at 25°C
Detection System for Manual, Xmatrx, NanoVIP & i6000 systems***	Use BioGenex Two-Step Super Sensitive™ Polymer-HRP IHC Detection System/DAB; see p. 2 for more information

*FB: positive control micro chamber slides, FG: positive control microscopic slides. Xmatrx & NanoVIP requires micro chamber slides.

***Pretreatment times will vary based on individual microwave power.

Category	Antibodies	Revision No.	B
Document No.	932-D50M-EN	Release Date	01-Jul-2024

***For automation systems (Xmatrx-Elite, NanoVIP & i6000 Diagnostics), refer to the factory protocols provided with the instrument.

Detection System	Two-Step HRP Kit	Link and Label Kit
Manual	QD440-XAKEN (1000 Test)	QP300-XAKE (1000 Test)
	QD430-XAKEN (1000 Test)	
	QD420-YIKEN (500 Test)	QP900-9LE (500 Test)
	QD400-60KEN (60 Test)	
Xmatrx - Automation	QD550-YCDEN (200 Test)	N/A
NanoVIP- Automation	QD551-YCDEN (100 Test)	N/A
i6000 - Automation	QD410-YAXEN (200 Test)	N/A

Precautions

This product contains sodium azide at concentrations of less than 0.1%. Sodium azide is not classified as a hazardous chemical at the product concentrations, but proper handling protocols should be observed. For more information, a Safety Data Sheet (SDS) for sodium azide is available upon request. Dispose of unused reagents according to Local, State and Federal Regulations. Wear suitable Personal Protective Equipment, do not pipette reagents by mouth, and avoid contact of reagents and specimens with skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with copious amounts of water.

Quality Control

Refer to BioGenex detection system documents for guidance on general quality control procedures.

Troubleshooting

Refer to the troubleshooting section in the documentation for BioGenex Detection Systems (or equivalent detection systems) for remedial actions on detection system related issues, or contact BioGenex Technical Support Department at 1-800-421-4149 or support@biogenex.com or your local distributor to report unusual staining.

Expected Results

This antibody stains membrane in positive cells in formalin-fixed, paraffin embedded tissue sections. An example image of a tissue section stained with this antibody can be found on the product page on the BioGenex website. Interpretation of the staining result is solely the responsibility of the user. Experimental results should be confirmed by a medically-established diagnostic product or procedure.

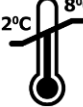
Limitations of the Procedure

Improper tissue handling and processing prior to immunostaining can lead to inconsistent results. Variations in embedding and

fixation or the nature of the tissue may lead to variations in results. Endogenous peroxidase activity or pseudo peroxidase activity in erythrocytes and tissue biotin may result in non-specific staining based on the detection system employed. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive with horseradish peroxidase systems. Improper counterstaining and mounting may compromise the interpretation of results.

Bibliography

1. Demeure, C.E., et al. 2001. T Lymphocytes infiltrating various tumour types express the MHC class II ligand lymphocyte activation gene-3 (LAG-3): role of LAG-3/MHC class II interactions in cell-cell contacts. Eur. J. Cancer 37: 1709-1718.
2. Andrae, S., et al. 2002. Maturation and activation of dendritic cells induced by lymphocyte activation gene-3 (CD223). J. Immunol. 168: 3874-3880.

	Temperature Limitation		In Vitro Diagnostic Medical Device
	Use By Date		Batch Code
	Non-Sterile		Consult Instructions for Use
	Representative in the European Community		Manufacturer

© 2020, BioGenex Laboratories. All rights reserved.

Category	Antibodies	Revision No.	B
Document No.	932-D50M-EN	Release Date	01-Jul-2024